OXYGEN & NEBULIZATION

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PRODUK AKL

Hospitech® Nasal Cannula 01
Hospitech® Oxygen Mask 02
Hospitech® Oxygen Tubing 03
Hospitech® Oxygen Mask with Reservoir Bag (Non-Rebreathing) 04
Hospitech® Oxygen Mask with Reservoir Bag (Rebreathing) 05
Hospitech® Disposable Nebulizer Set with Oxygen Mask and 82-Inch Tubing 06

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HOSPITECH™

HEALTHCARE FOR HUMANITY

Oxygen Mask with Reservoir Bag (Non-Rebreathing)

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HOSPITECH™

HEALTHCARE FOR HUMANITY

Oxygen Mask with Reservoir Bag (Rebreathing)

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HOSPITECH™

HEALTHCARE FOR HUMANITY

Disposable Nebulizer Set with Oxygen Mask & 82-Inch Tubing

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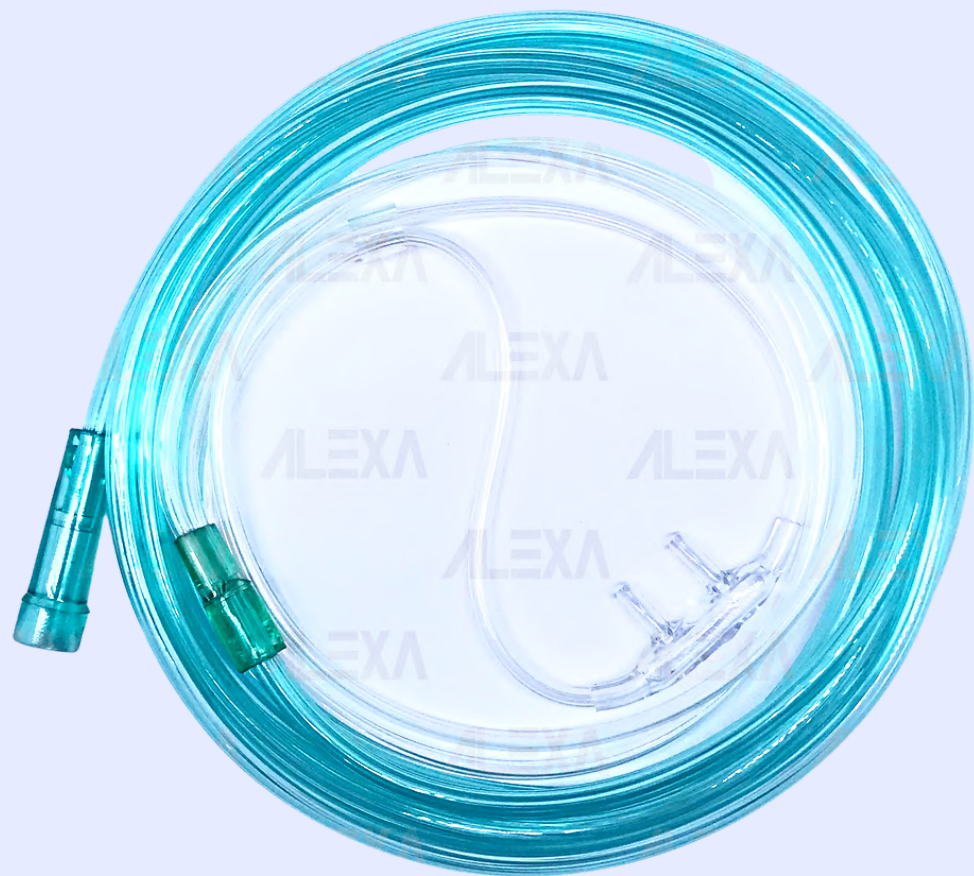
Nasal Oxygen Cannula

UNICORN
Nasal Oxygen Cannula

Direction For Use :
 1. Attach the oxygen supply tubing to the gas source and set the oxygen to the prescribed flow.
 2. Check for gas flow throughout the device.
 3. Insert the nasal tips into the nostrils with the two plastic tubes over the ears and under the chin.
 4. Gently adjust the plastic slide until the cannula is secure.

Caution :
 * For single use. Discard after use.
 * Do not use if package is opened or damaged.
 * Do not store at extreme temperature and humidity. Store in cool and dry place.
 * Ensure that all connections are secure and air circulates freely through the tubing.

Adult
 Child
 Infant
 Neonate

MFG :
 EXP : MAR 2022
 MAR 2027
 LOT : 21220323

KEMENKES RI AKD 10403122426
 Produced By : PT. Ocean Medika Indonesia
 Tangerang, Banten - Indonesia
 15214
 Distributed By : PT. Alexa Medika
 Medan - Indonesia

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